# Chabner XRT Radiation Bra

---

Instructions for Use

CE

# LOT

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LOT
批次
批次
BROJ
ŠARŽE
PARTI
ALB
ERÄ
LOT
CHARGE
ΠΑΡΤΙΔΑ
LOT
LOTTO
ロット
로트
PARTIJA
PARTI
SERIA
LOTE
LOTE
LOT
ПАРТИЯ
ŠARŽA
LOT
LOTE
PARTI
ລຸກ
LOT

**INDICATIONS FOR USE / INTENDED USE**

- The device is indicated to aid in supporting adolescent and adult patients undergoing radiation therapy including electron, photon, and proton treatments. The device is also used during image acquisition to support treatment planning.
- The device is not intended for use with patients under 12 years of age.

These Instructions apply to the use of the following Chabner XRT Radiation Bra and accessory Extenders:

Chabner XRT Radiation Bra Size 01	CRTBRA01
Chabner XRT Radiation Bra Size 02	CRTBRA02
Chabner XRT Radiation Bra Size 03	CRTBRA03
Chabner XRT Radiation Bra Size 04	CRTBRA04
Chabner XRT Radiation Bra Size 05	CRTBRA05
Chabner XRT Radiation Bra Size 06	CRTBRA06
Chabner XRT Radiation Bra Size 07	CRTBRA07
Chabner XRT Radiation Bra Size 08	CRTBRA08
Chabner XRT Radiation Bra Size 09	CRTBRA09
Chabner XRT Radiation Bra Size 10	CRTBRA10
Chabner XRT Radiation Bra Size 11	CRTBRA11
Chabner XRT Radiation Bra Size 12	CRTBRA12
Chabner XRT Radiation Bra Size 13	CRTBRA13
Chabner XRT Radiation Bra Extender, Shoulder	CRTSHLDEXT
Chabner XRT Radiation Bra Extender, Back	CRTEXT

**CAUTION**

Federal (United States) law restricts this device to sale by or on the order of a physician.

**PRECAUTION**

Store at temperatures of less than 35 °C (95 °F) and below 90% condensing humidity.

**MRI SAFETY INFORMATION**

MR Safe

- Devices are MR Safe.
- In non-clinical testing, the image artifact caused by the device extends up to 3mm from the seams of the bra back panels and extender when imaged with an axial gradient echo pulse sequence and a 3.0 T MRI system.

**WARNING**

- Do not use if device appears damaged.
- Verify all angles of treatment, attenuation characteristics and WET values prior to treating patients. Refer to Technical Data Sheet at [WWW.CHABNERXRT.COM](http://WWW.CHABNERXRT.COM) for WET values.
- When positioning patient for first time, use setup sheet to record all adjustments. Setup sheet is available at [WWW.CHABNERXRT.COM](http://WWW.CHABNERXRT.COM).
- Verify patient position with completed setup sheet prior to treatment.
- Verification image should be taken to confirm patient position prior to treatment.
- Device should be used for a single patient throughout setup and treatment cycle.
- If new device required, re-simulate patient.
- Cutting portals in device may compromise structural integrity. Verify structural integrity prior to use.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

**MEASUREMENT AND SIZING**

- NOTE:
- For ease of measurement, take patients measurement at their side.
  - Measure with patient's arms and shoulders relaxed.

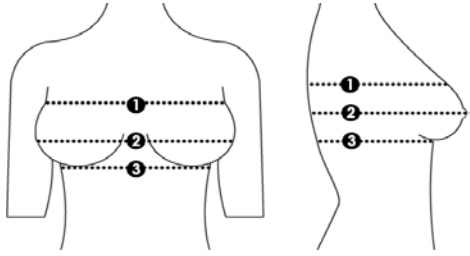
- It is recommended that patients with pendulous breasts wear personal bra during measurements.

**Step 1:** Upper Chest Measurement - Measure patient tightly across upper chest, under arm at arm pit level and around back.

**Step 2:** Bust Measurement - Measure patient loosely around fullest part of bust, under arm and around back.

**Step 3:** Band Measurement - Measure patient tightly under bust, across ribcage and around back. Note final measurement to reference on chart below.


NOTE: If band measurement exceeds 48in (122cm), utilization of back extender (s) will be necessary.

**Step 4:** Subtract upper chest measurement (*step 1*) from bust chest measurement (*step 2*). Note final measurement to reference on chart below.

**Step 5:** Select Chabner XRT® size on chart below by using step 3 and 4 from above. Use of back extender and/or overlapping of back panels may be necessary.


		Step 4 (inches)												
		1	2	3	4	5	6	7	8	9	10	11		
Step 3 (inches)	28		1	1	2	2	3	4	5					71
	30	1	1	2	2	3	4	5				9	76	
	32	1	2	2	3	4	5		7		9	9	81	
	34	2	2	3	4	5	5	7	7	9	9	10	86	
	36	2	3	4	5	5	7	7	9	9	10	10	91	
	38	3	4	6	6	7	7	9	9	10	10	11	97	
	40	4	6	6	7	7	9	9	10	10	11	11	102	
	42		6	8	8	9	9	10	10	11	11	12	107	
	44		8	8	9	9	10	10	11	11	12	12	112	
	46		8	9	9	10	10	11	11	12	12	13	117	
48		9		10	10	11	11	12	12	13	13	122		
		Step 4 (cm)												
		3	5	8	10	13	15	18	20	23	25	28		

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-  Chabner XRT® with Overlapping Back Panels
-  Chabner XRT® Radiation Bra
-  Chabner XRT® Radiation Bra with Back Extender\*

\*NOTE: For extended band measurement, Chabner XRT® Bra can be used with a Chabner XRT® Back Extender. (Contact CHABNER XRT sales representative to order a Chabner XRT® Back Extender, if necessary.)

**CHABNER XRT®FITTING**

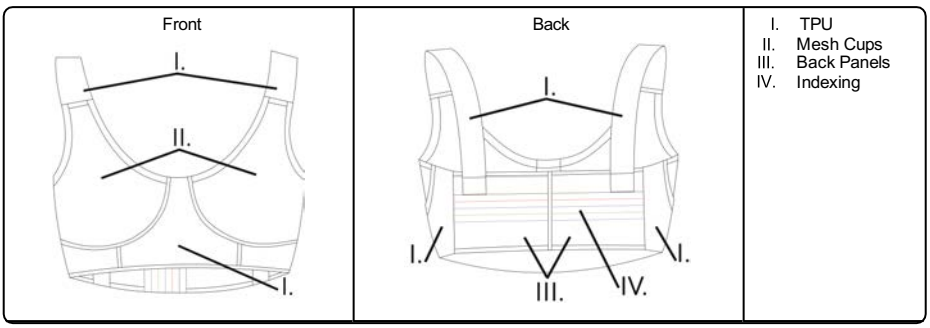
NOTE: Chabner Fitting Bras may be purchased by contacting your CHABNER XRT sales representative.

1. Once appropriate Chabner XRT® Radiation Bra size has been selected for patient, ensure bra fits properly as follows:
  - a. Chabner XRT® Radiation Bra should fit snug.
  - b. All breast tissue, including lower pole and lateral aspects, should fit entirely in mesh cup.
  - c. If breast tissue extends beyond mesh cup, adjust Chabner XRT® Radiation Bra as necessary or select one size larger.
  - d. If breast tissue does not fill mesh cup, adjust Chabner XRT® Radiation Bra or select one size smaller.
  - e. Adjust shoulder straps to elevate breasts and eliminate folds at the inframammary or lateral aspects.

2. Further adjustments to reach proper fitting may be needed. See possible options below.

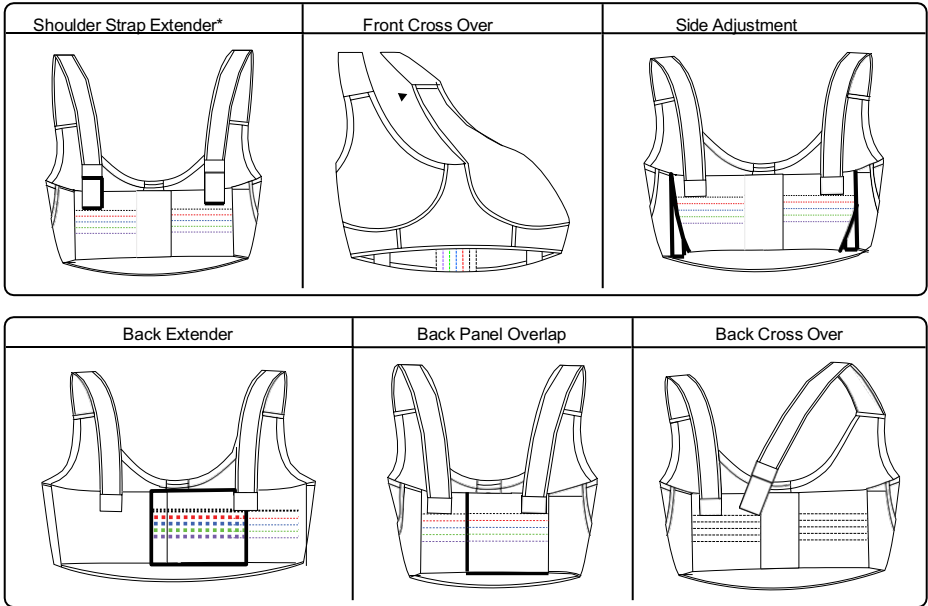\* NOTE: Shoulder Strap Extenders may be purchased through your CHABNER XRT sales representative.

3. Position patient as required for intended radiological scan or treatment.
4. With an indelible marker, draw a line at Velcro® attachment points (*shoulder, lateral and posterior*) for reference at future Chabner XRT® Radiation Bra wearing(s).
5. Once properly fitted, do not move shoulder or lateral Velcro® adjustments.
6. To take Chabner XRT® Radiation Bra on and off, use posterior Velcro® attachment.

**REPROCESSING**

**⚠ WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

**Chabner XRT® Radiation Bra**

- Device can not be reprocessed.
- Integrity of bra will become compromised if cleaned.
- If device becomes soiled, discard and proceed with new unit. Re-simulate patient as necessary.

**Fitting Bra**

- If device becomes soiled and unable to be cleaned by washing, discard and proceed with new unit.

**Fitting Bra**

1. Fitting bra can be hand-washed or washed on a delicate cycle on cold with a gentle detergent.
2. Air dry fitting bra.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CHABNER XRT IS A REGISTERED TRADEMARK
OF CHABNER XRT. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE
**Chabner XRT, LLC**